
PILOT-19 LITE QUICK START GUIDE

IMPORTANT!

- Fully charge the battery before storage and recharge it once every 6 months while stored! Failure to do so will void the warranty.
- DO NOT expose the battery to temperatures above 40°C / 104°F or below -10°C / 14°F. Extreme temperature ranges can cause damage to the battery cells and result in fire Hazards!

PILOT-19 LITE CONTROL PANEL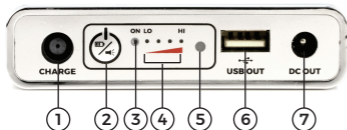

1. Charge Port
2. Multi Function Button
3. Blue "ON" Indicator Light
4. Green Charge Level Indicator Lights
5. LED Flashlight
6. USB Port (5V 2.5A Out)
7. DC Out Port

CHARGING THE PILOT-19 LITE

Charge times for the Pilot-19 Lite via the React Health Luna TravelPAP device AC adapter is approximately 4-5 hours.

Charge time via other compatible power adapters may vary. The Pilot-19 Lite can also be charged using the Medistrom Pilot Lite AC Adapter (P24ACCACA).

NOTE: The Medistrom Pilot Lite AC Adapter (P24ACCACA) should not be used in the backup configuration.

Please fully charge your Pilot-19 Lite before using for the first time.

CONNECTING PILOT-19 LITE TO PAP DEVICE

Plug the long L-shaped yellow tipped end of the DC Output cable for React Health Luna TravelPAP (P19BM1DOC) into the DC Out Port of your Pilot-19 Lite and plug the short L-shaped yellow tipped end of the DC Output cable for React Health Luna TravelPAP (P19BM1DOC) into the React Health Luna TravelPAP device.

To use in backup configuration, keep battery charging while connected to PAP device.

For technical support and questions about this product please email to support@medistrom.com or contact us by phone:

North America: 1-888-407-4791 | Australia: 1300-146-358 | Europe: +44-800-060-8576

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